**Schlauchadapter für Fremdbefeuchter/  
Slangadapter voor externe bevochtiger/  
Tube adapter for third party humidifiers/  
Adaptateur de tuyaux pour humidifacateurs WM 23366  
tiers/  
Adattore per tubi di umidificatori di altri  
produttori**

**Montageanleitung /  
Montagehandleiding/  
Assembly instructions/  
Notice de Montage/  
Istruzioni di montaggio**

**WEINMANN**  
medical technology

**WEINMANN**  
medical technology

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### Toepassing

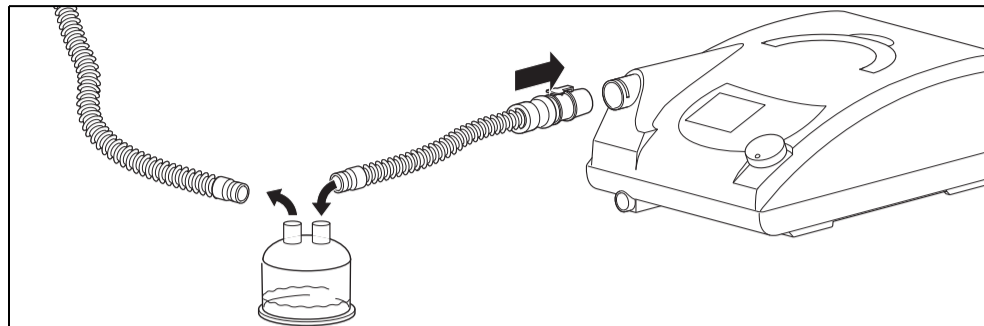
## Intended use

The tube adapter allows humidifier chambers HC 300 and MR 290 from Fisher & Paykel to be used with the following WEINMANN therapy devices: VENTImotion, VENTIllogic, VENTIllogic LS (if leakage ventilation is used) and VENTIllogic plus (if leakage ventilation is used).

## Safety instructions

- Use the tube system only in combination with the above-mentioned devices.
- Follow the instructions for use for the ventilator and respiratory air humidifier used.

## Assembly

1. Connect the tube adapter to the ventilator and the humidifier chamber.
2. Connect the humidifier chamber to the breathing mask. You can use creased tube WM 24445 to do this.
3. Perform a function check as described in the instructions for use for the therapy device used.

## Hygiene treatment

Perform a function check as described in the instructions for use for the therapy device used.

## Technical data

If you use third-party humidifiers, the technical data quoted in the instructions for use for the therapy device will change. The table below lists the differences which will result when third-party humidifiers are used.

	VENTImotion / VENTIllogic		VENTIllogic LS / VENTIllogic plus	
	HC 300	MR 290	HC 300	MR 290
Maximum flow	280 ± 15 l/min	255 ± 15 l/min	270 ± 15 l/min	250 ± 15 l/min
Precision of volume measurement	± 22 %		± 22 %, at least 25 ml	
System resistance at an air flow rate of 60 l/min at the patient connection opening	0,21 $\frac{\text{kPa} \cdot \text{s}}{\text{l}}$	0,25 $\frac{\text{kPa} \cdot \text{s}}{\text{l}}$	0,26 $\frac{\text{kPa} \cdot \text{s}}{\text{l}}$	0,30 $\frac{\text{kPa} \cdot \text{s}}{\text{l}}$

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