

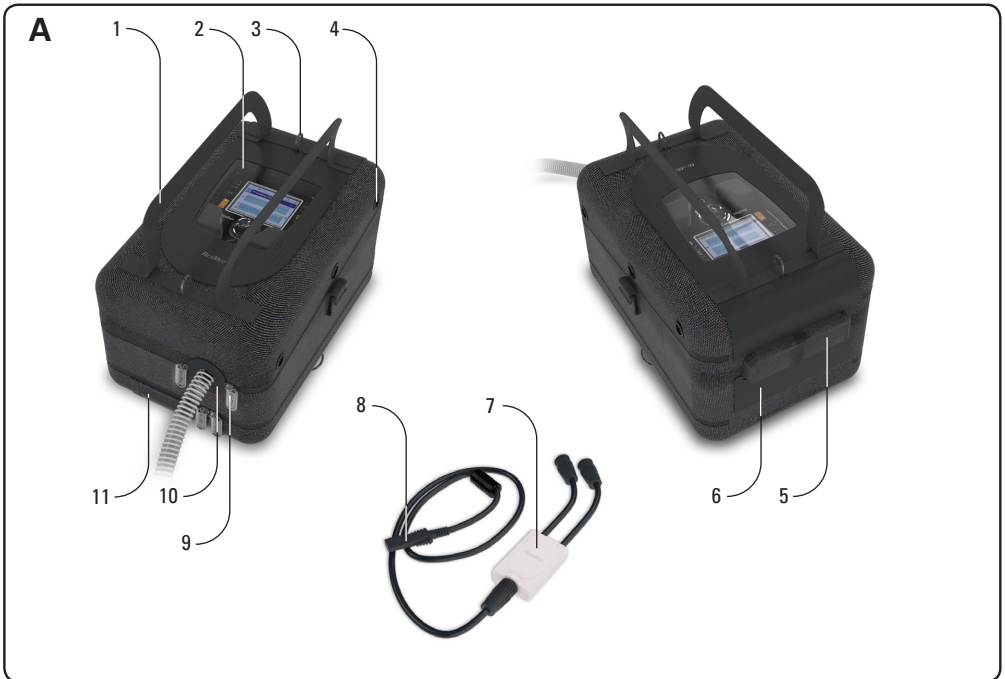
RESMED

Mobility bag for Stellar™

User Guide

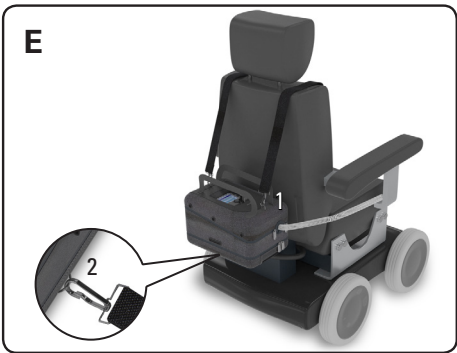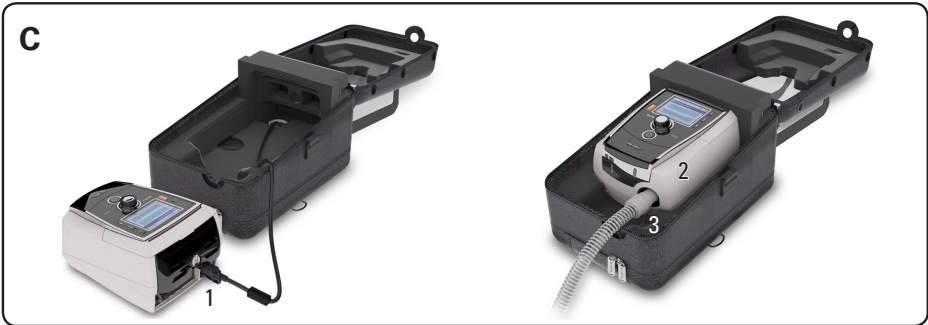English • Deutsch • Français • Italiano • Español • Português •
Nederlands • Svenska • Dansk • Norsk • Suomi •
简体中文 • 繁體中文

Introduction

The mobility bag is intended for mobile use of the Stellar. It protects the device against dust and water. The straps allow the mobility bag to be fixed to a wheelchair or to be hung on a shoulder. For longer mobile use, power the device using the ResMed Power Station II external power supply unit or other validated and approved DC power supply that meets the required voltage and current specifications stated in the Stellar User Guide.

Description of the mobility bag

Refer to illustration A.

1. Carrying handles.
2. Protective flap.
3. Carrying straps hook.
4. Vents (several).
5. Air inlet.
6. Access flap to Stellar connections.
7. Coupler for ResMed Power Station II.
8. DC cable (DC Connection Stellar).
9. Zipper.
10. Air tubing sealing straps.
11. ResMed Power Station II charging status window.

Setup

Setup for use with ResMed Power Station II

Refer to illustration B.

1. Connect the coupler to the DC cable.
2. Attach the coupler in the mobility bag.
3. Pull the DC cable through the hole of the bottom of the ventilator compartment.
4. Place the two batteries in the mobility bag.
5. Connect the batteries with the coupler cables.

Refer to illustration C.

1. Connect the DC cable to the DC power socket of the Stellar.
2. Place the Stellar in the mobility bag.
3. Attach the air tubing and the antibacterial filter (if applicable) to the air outlet of the device.

Closing the mobility bag

Refer to illustration D.

1. Close the zippers.
2. Secure the air tubing with the VELCRO® straps.

Setup for use with a wheelchair

Refer to illustration E.

1. Attach the carrying strap to the carry hooks on the top of the mobility bag.
2. Lay the carrying straps over the back of your wheelchair and attach it to the hook on the bottom of the mobility bag.

Alternatively you can hang the mobility bag on your shoulder or on your back.

Using the pulse oxymetry sensor

Refer to illustration F.

1. Attach the pulse oximetry sensor to the Stellar.
2. Close the flap.
3. Attach the finger pulse sensor to the pulse oximeter.

Using the device

To operate the device open the protective flap for the dial protection.

Cleaning and maintenance

The cleaning and maintenance described in this section should be carried out regularly. Refer to your device, mask and other accessories user guide for detailed instructions for care and maintenance.

Wipe the interior and exterior of the mobility bag with a damp cloth and mild detergent. Leave the cleaned parts to dry.

Technical specifications

Dimensions (L x W x H): 13" x 8.3" x 7.9"
330 mm x 210 mm x 200 mm
Protection class: IPX2

General warnings and cautions

- Read the entire manual before using the mobility bag.
- Read the Stellar User Guide and when using accessories, read the manufacturer's User Manual.
- Do not cover any air inlets.
- Always close the zippers and the flaps to protect the device from moisture.
- Oxygen must not be used while the device is being operated within the mobility bag.
- When using Stellar in the Mobility Bag, check the alarm volume is loud enough to be heard in the noisiest environment in which it will operate.

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